



**new
product**

slee *solutions
for
pathology*

TISSUE PROCESSORS

CELLUTEC PRO

CELLUTEC PRO +

VACUUM PROCESSING /
FASTEST NON-MICROWAVE TISSUE PROCESSING
STATE OF THE ART FEATURES
SAFE AND HIGHLY RELIABLE

DESIGN &
MANUFACTURING
MADE IN GERMANY

Cellutec Pro / Pro + Vacuum / non-microwave tissue processor

*Presenting the advanced tissue processor **Cellutec Pro / Pro +**, Slee hits another milestone. The standard version **Cellutec Pro** combines highest processing capacity with reliability and intuitive operation. The Slee vacuum tissue processor **Cellutec Pro +** is the fastest non-microwave tissue processor available today, providing additional speed through a flow-through reagent heating system (EHE) for faster heat up of reagents during filling process. Up to 420 biopsies can be safely processed in 45 minutes.*

The Cellutec Pro + features a patented device called EHE (Enhanced Heat Exchanger): The reagents are heated safely and without delay during filling into the sample processing chamber. This procedure ensures successfully and rapidly processing of biopsies.

Every sample is evenly warmed at the desired temperature from the very beginning of each processing step. Thanks to the EHE's special design and multiple safety features, any risk of overheating is being avoided.

Two functionalities – one device

Vacuum tissue processor	Fastest non-microwave tissue processor available
Vacuum / pressure heating and agitation of reagents	additional EHE for faster tissue processing

Highlights

- Fast processing (microwave-free)
- Reagent management system
- Remote fill & drain
- 420 samples capacity
- 15" touch screen operation control
- 3 level sensors
- Reagent partial load capacity
- Wax cleaning cycle
- Data backup on USB port
- 18 processing programs
- Reverse processing special program
- Process end time management
- Fume control charcoal filter
- Anti-blackout and power failures devices
- Remote alarm by SMS
- Remote monitoring and servicing
- Xylene-free processing

Item No.

Cellutec Pro 115 - 230 V, 50/60 Hz

Vacuum tissue processor

11000700

new

Cellutec Pro + 115 - 230 V, 50/60 Hz

Fastest non-microwave tissue processor

11000710

new





Backlit bottles to visualize the reagent quality

Standard components	Pro	Pro +
Sample processing chamber	✓	✓
Sample baskets (3 pcs. à 150 cassettes)	✓	✓
Reagent tanks 5 l capacity (12 pcs.)	✓	✓
Paraffin containers 5 l capacity (3 pcs.)	✓	✓
EHE for fast processing		✓
Backlit bottles		✓
NFC (Near Field Communication) technology		✓
LIS integration capability	✓	✓
SPC level sensors (3 pcs.)	✓	✓
Charcoal filter	✓	✓
Color touch screen 15" with 3 cables (VGA-USB touch- screen - power 12 V)	✓	✓
Mains cable	✓	✓
Manual	✓	✓

Optional accessories	Item No.
RFID reader for user identification	11000720
ALIS (connection with LIS system De- dalus Armonia)	11000721
OLIS (connection with other LIS sys- tems)	11000722
UPS (uninterruptable power supply)	11000723
Barcode Reader Gun	11000724
More on request	

Specifications	Pro / Pro +
Processing capacity	up to 420 standard cassettes
Remote fill & drain ports	2
Wax dumping ports	1
Processing programs	18
Purge reagents	2
Reagents temperature range	20 - 65 °C
Paraffin temperature range	55 - 65 °C
Process start delay time	up to 2 weeks
Reagent partial load capacity	yes
Reagent management system (RMS)	yes
Remote fill & drain (RFD)	yes
External wax dumping (EWD)	yes
USB ports	5
Remote alarm ports	1
Network ports (Ethernet)	2
Approvals	CE / IVD / UL
Available software languages	CN, CZ, EN, FR, DE, HU, IT, KZ, PL, PT, RO, RU, ES, TR
Electrical parameters	115 - 230 V, 50/60 Hz
Dimensions (w x d x h), with monitor	680 x 700 x 1500 mm
Weight w/o accessories	226 kg

1 port for
external wax
dumping

2 ports for
reagent remote
fill & drain

Intuitive graphic
user interface

Sample processing
chamber

USB port

5 l reagent tanks



Display

User-friendly graphics and intuitive GUI (Graphic User Interface). The 15" touchscreen with animated reagent flow leads to a completely new user experience.

Integrated ports

Fast connectors for RFD and EWD fill and drain system: 2 ports for easy and automatic reagent remote fill & drain, 1 port for external wax dumping.



Sample processing chamber (stainless steel)



- Safe and easy to operate lid lock mechanism
- Special, easy to clean, surface treatment
- Gas piston to keep the lid open
- Lid open / close sensor
- Fume aspiration system

Fastest non-microwave tissue processor



Cellutec Pro + safely processes up to 420 biopsies in 45 minutes. Vacuum / pressure, heating and agitation of reagents to gently and safely process the biopsies.

5 l reagent tanks, Xylene-free



Large tanks, work Xylene-free: All Slee tissue processors work efficiently with Xylene substitutes. The Cellutec Pro + tanks are highlighted from the back with an LED light.

RMS (Reagent Management System)



Reagent management and quality control system with up to 18 programmable processing protocols. Quick start of the most used protocols.

PWD (Protected Wax Dumping)



For better operator protection, it is possible to drain the used paraffin directly into tank number 10 manually or automatically during the process.



SLEE medical GmbH • Am Neuberg 14
55268 Nieder-Olm • Germany
www.slee.de

T: +49 (0) 6136 76997-0

E: mail@slee.de



www.slee.de



[YouTube](#)



[LinkedIn](#)



[Instagram](#)